



**Model Number
ATN1XX-XXX**

www.norix.com • TEL 800 234-4900

Assembly Notes: Norix is not responsible for any damage due to improper installation. If your shipment does not match the parts list, please contact Norix Customer Service at 800-234-4900. Norix reserves the right to modify design specifications at any time without an obligation to owners of previously sold products.

Parts List				Tools Needed
ITEM	QTY.	PART NUMBER	DESCRIPTION	
A	1	ATN1XX-XXX	ATTENDA BED	Low RPM Variable Speed Drill
B	1	ATN-R06N	KIT, ATTENDA RESTRAINT RING - SET OF 6	3/8" Concrete Drill Bit
				Driver for Anchor
				Pin in Torx® T40 Driver

Step 1

Place the bed in the desired location. Mark the floor through each hole and move the bed aside.

Step 2

Prepare the holes for anchoring according to the manufacturers instructions.

Step 3

Move the bed back into position. Note that the underside of the bed has a groove molded all around the base of the bed. This groove is intended for caulking material if desired. If caulking is desired, use a pry bar and lift the edge of the bed slightly and place the caulking between the bed and floor at the location of the caulk groove.

Step 4

Once the anchors (not supplied) have been set for the Attenda Bed (A) and the bed has been set into place, place the Restraint Rings (B) over the anchors, then fasten the bed and rings down using the hardware included with the concrete anchors (not supplied by Norix).

Step 5

Set the anchors according to the manufacturers instructions.

